FR	Notice d'utilisation	2
NL	Gebruikersaanwijzing	13
EN	User manual	24
DE	Gebrauchsanweisung	34
ES	Manual de instrucciones	45
IT	Istruzioni per l'uso	55
PT	Manual de utilização	65

FR
RECTO

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Eclusier 2

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EN REVITALIZING BELT “LE ROGER”

Le Roger User Manual

Before using Le Roger, please take a few moments to read this manual.

Device identification

Trade name: Le Roger

Product reference: CR1-BS-511

 Class I medical device – non-sterile

 05419980643711

Safety instructions

Read these instructions and safety instructions carefully. Failure to follow the warnings and safety instructions may result in personal injury or property damage.

Keep the warnings and safety instructions for future reference.

WARNING - Precautions for use

To ensure optimal use and proper functioning, this product must be used in accordance with the following instructions. Le Roger is intended solely for the purpose described in this user manual. The manufacturer cannot be held liable for damage caused by incorrect or careless use.

Indications for use

Le Roger is intended for men with venous or mixed erectile dysfunction who

wish to improve the rigidity and duration of their erections through mechanical stimulation of the perineum.

The device is for individual use only and may be used in a private setting or as part of perineal rehabilitation under medical supervision.

Contraindications

Do not use in the following cases:

- known allergy to latex or any component of the product,
- skin condition or injury in the contact area,
- severe circulatory disorders, local inflammation or urogenital infection.

If in doubt, seek advice from a healthcare professional. Any serious incidents must be reported to Roger srl via sav@leroger.com and to the relevant local authorities, who will provide you with further advice.

Cautions and warnings

Do not use under pressure for more than 1 hour (60 minutes).

Do not use Le Roger if it shows signs of damage (tears, cuts, etc.).

Discontinue use if you experience pain or discomfort.

This product is not a contraceptive.

It is advisable to apply a lubricating gel to hairy parts of the body or to shave to avoid discomfort. Only use water-based gels to keep Le Roger in good condition.

Keep out of reach of children

EN
RECTO

FRONT straps **1**

Lock **2**

Opening to allow
the genitals to pass
through **3**

4 Constrictor

5 REAR straps

6 Tightening buckles and
comfort pads

Opening to allow
the genitals to pass
through **3**

Activator **7**

Constrictor **4**

5 REAR straps

8 Inserts for REAR straps

EN

1 FRONT straps

Description

Le Roger is a flexible revitalising belt made from a combination of medical-grade silicone and elastic fabric.

It applies localised pressure to the perineum to restrict venous return and thus support the natural mechanism of erection.

The device works mechanically, non-invasively and without the use of chemicals.



Composition

Le Roger consists of:

- a ventral section with an oval opening **(3)** to allow the genitals to pass through,
- a lock **(2)**
- an activator allowing localized pressure **(3)**
- a constrictor **(4)**,
- two front straps **(1)**,
- two rear straps **(5)**, ending in tightening buckles and comfort pads **(6)**.

As part of its ongoing product improvement program, the manufacturer reserves the right to modify the technical specifications and design of the product at any time without specific notice.


FITTING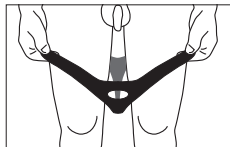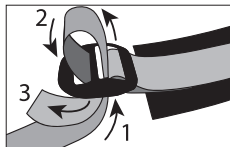

1 - Preparing the straps: Assemble the straps without crossing them. Pass the free strap through the buckle from underneath, through the central opening. Pull it out through the second opening (back and forth movement)

2 - Putting on Le Roger: Place the front part with the opening **(3)** in front of you. Put on Le Roger like underwear. Check that the activator **(3)** is positioned against the body.

3 - Positioning the genitals: Insert the testicles and penis into the opening **(3)**.

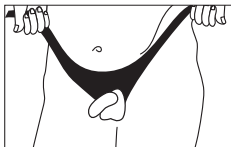4 - Adjusting the tightness:

Position the straps **on the hips**.

Tighten both sides evenly to keep the perineal band centered. Le Roger should exert firm, centered pressure on the perineal area.

How it works

Erection is optimal when the pressure exerted on the body is at its maximum.

This pressure can be adjusted by tightening the straps using the buckles.

Erections can become painful if this pressure is excessive.

Le Roger device develops a natural erection that follows the natural human process: fantasies and light stimulation.

Removal

- Gently loosen the closure provided by the tightening buckles
- Free the genitals from the abdominal part
- Slowly remove the device

Cleaning and maintenance

Taking care of the device will increase its lifespan. Clean Le Roger before and after each use with soap and warm water. Do not use alcohol-based cleaners, bleach, or abrasive detergents. Wipe it dry and leave it to air dry. Do not iron. Store it in a dry, clean place away from light and heat. Avoid prolonged contact with silicone-based oils or lubricants. Store it in a dry, clean place away from heat.

Disposal and recycling information

Dispose of Le Roger in household waste. Do not dispose of on public roads or in the environment.

WARRANTY

Le Roger is guaranteed for 2 years.

Warranty conditions

This product is covered by a warranty against manufacturing defects, the duration of which depends on the legislation in force in each country (2 years in the European Union).

If you need to make a warranty claim, please contact Roger srl customer service. If you need to ship the product, please indicate the defect and include a copy of the proof of purchase.

The warranty terms and conditions are as follows:

1. Roger products are covered by a two-year warranty from the date of purchase. In the event of a warranty claim, proof of purchase must be provided in the form of a receipt or invoice.
2. During the warranty period, defects related to material or manufacturing errors will be covered free of charge (excluding shipping costs).
3. Services performed under warranty do not extend the warranty period for either the Product or the replaced components.
4. The following are excluded from the warranty:
 - a. Any damage due to improper use, for example, failure to follow the user manual.
 - b. Damage due to repairs or interventions carried out by the purchaser or unauthorized third parties.
 - c. damage occurring during transport of the device from the manufacturer's site to the user or during shipment of the device to customer service.
 - d. accessories subject to normal wear and tear.
5. We accept no liability for consequential damage caused directly or indirectly by the product, including when the damage to the product is covered by the warranty.

Customer service

info@leroger.com

This product is strictly for adults only. Keep out of reach of children.

Distributed by Roger srl, 24 rue de la Lasne, 1380 Lasne, Belgium

Roger srl® and Le Roger® are registered trademarks. All rights reserved.

The design of the Le Roger belt is a work of art and is protected by copyright.

For more information, visit **www.leroger.com**

Last revision: 11/2025 – Version 1.1.

Batch: see packaging.

Please keep proof of purchase and the lot/SN label for the duration of the warranty.

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